

Travel and Airport Security

The NESS H200 Wireless System charger is compatible with Australian, U.K., European Union, and U.S. voltages: 110/220 V, 50/60 Hz.

Turn off your NESS H200 Wireless System before going through airport security. Wear loose clothing so that you can easily show the security person your H200 Wireless System. The NESS H200 Wireless System will likely set off the security alarm. Be prepared to remove the NESS H200 Wireless System so that security can scan it, or ask for the system to be scanned if you do not want to remove it. You may want to carry a copy of your H200 Wireless System prescription. A prescription can be useful when passing through customs as well.

To request a copy of your prescription, call the Bioness Client Relations Department: Telephone: (800) 211-9136, Option 3; or (661) 362-4850, Option 3. A Bioness representative can fax or mail you a copy.

Note: The NESS H200 Wireless System contains radio transmitters. The Federal Aviation Administration rules require that all radio-transmitting devices be turned off during flight.