

Travel

The NESS L300 charger is compatible with both European and U.S. voltage: 110/220 V, 50/60 Hz. A voltage converter is not needed. A simple outlet adaptor should work. Outlet adaptors can be purchased at small electronic equipment stores.

Turn off your L300 system before going through airport security. Wear loose clothing so that you can easily show the security person your NESS L300. The NESS L300 will likely set off the security alarm. Either ask for a “hand scan” or be prepared to remove your NESS L300 so that security can scan it. You may want to carry a copy of your NESS L300 prescription. A prescription can be useful when passing through customs as well.

To request a copy of your prescription, call Bioness Customer Support: telephone: (800) 211-9136, Option 2; or (661) 362-4850, Option 2. A Bioness representative can fax or mail you a copy.

Note: The NESS L300 contains radio transmitters. The Federal Aviation Administration (FAA) rules require that all radio-transmitting devices be turned off during flight.